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HAND-OSTEOSYNTHESE-SYSTEM

The Micromed Hand-Osteosynthese-System features highest quality in material and accomplishment of the implants. Major fields of application are osteosynthesis in traumatology, reconstructive surgery as well as replantation of the phalanx of fingers. Furthermore, our systems are excellently suitable for therapeutic surgery for treating rheumatic diseases.

THE PLATES

We offer application-oriented plates of different thicknesses. XS- and S-Fixation plates of 0.6 mm thickness for treatment of middle phalanx fractures.

M-Fixation plates and M-Compression plates of 1.0 mm thickness for treatment of proximal phalanx fractures and metacarpal IV fractures.

L-Compression plates of 1.3 mm thickness for treatment of metacarpal I-, II-, III-, and V fractures.

THE INSTRUMENTS

The clear basic main instrumentation facilitates the use of plates and screws with ergonomic forms and colour coding.

Colour code ● for XS instruments

Colour code ● for S instruments

Colour code ● for M and L instruments

Instruments which are used for several implant sizes do not carry any colour code.

STERILE IMPLANTS

All implants are available in non-sterile and sterile packaging. To order sterile products add an "S" at the end of the article number (e.g. 400-012-140S).

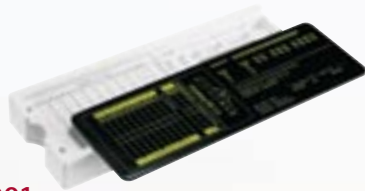
HAND SURGERY

Hand surgery, originating from war surgery, is a discipline which arose from the necessity of interdisciplinary treatment of the upper extremity. This field developed from traumatology, neurosurgery, and plastical surgery.

460-012-001

Schraubendreher Handgriff, drehbar, 1,2 mm
Screwdriver handle, rotatable, 1.2 mm

380-012-001

XS Implantatbehälter, 1,2 mm, leer
XS Implant Tray, 1.2 mm, empty

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LOCKING SYSTEM FOR OSTEOSYNTHESIS PLATES

The patented plate-screw-locking system allows for a multi-axial screwing into the bone with fixed-angle fixation of screw and plate.

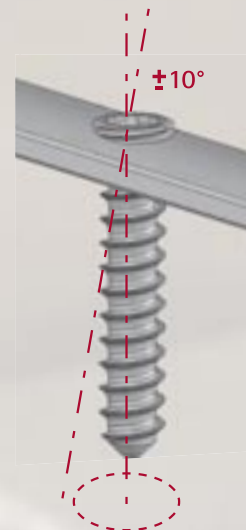
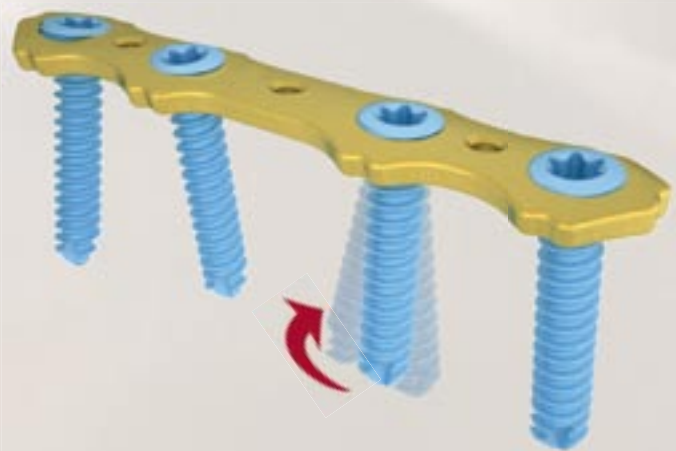
The functioning is based on a plastical forming between head thread of the screw and the artricks of the plate hole while screwing. Repeated use of the locking, that is, screwing in and out, several times, is possible.

THE PLATES

SW fixed-angle plates, plate thickness 1.0mm for 2.0 mm FlexFix screws

SW fixed-angle plates, plate thickness 1.5mm for 2.4 mm FlexFix screws

Knochenplatten mit winkelstabilem Verriegelungssystem *Osteosynthesis plates with fixed-angle locking system*

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- Osteotomien
- Frakturen

ADVANTAGES

- Locking function from 1.0 mm plate thickness onwards
- Multi axial, fixed-angle, 10° multi directional
- No additional components necessary

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APPLICATION OF SCREWS

The application of the Micromed locking system takes place in two steps:

Step 1

Twisting the screw into the bone until the head thread of the screw touches the outline of the plate cavity.

Step 2

Twisting the screw head into the plate cavity (-locking-) until the screw head and plate surface sit flush with each other.

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PICO-PIN-COMPRESSION SCREW

Pico-Pin is a new compression screw without canalisation and with integrated wire guidance. This serves as guiding wire which is preferably used with ordinary Herbert Screws.

Due to the innovative design, Pico-Pin allows for smaller diameters compared to Herbert Screws. This is a major advantage in the case of arthrodesis of smaller bones, such as the finger joint.

Using ordinary screws for arthrodesis comprises the risk of harming the onychostroma up to bone fractures. Due to the diameter reduction of the head thread, risk is reduced significantly.

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ADVANTAGES

- Risk reduction in the context of harming onychostroma and bone fractures
- Small diameter of screw head (Ø 2.5 mm)
- Small diameter of screw shaft (Ø 1.8 mm)
- Small pin diameter (Ø 1.25 mm)
- Self-boring tip
- No Kirschner wire necessary
- Applicable in minimally invasive surgery

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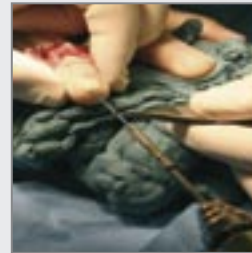
Resection of the arthritic joint surface in corresponding base angulation.

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Preparation of the bone for optimum arthrodesis results.

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Mechanical insertion of the Pico screw with motor system.

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Controlled application of compression through manual end fixation.

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SLIDING PLATE

IMPLANT FOR THE FOUR-CORNER-FUSION

The Sliding Plate is an innovative implant for partial midcarpal arthrodesis of the wrist ("Four-Corner-Fusion").

For pseudo-arthrosis of the Os Scaphoid and aged SL-band ruptures (SLAC and SNAC wrist) this therapy has established itself as treatment strategy. The application of a compression between the to be merged carpal elements hereby benefits a complete arthrodesis. The Sliding Plate offers the possibility to apply the compression centrally by means of turning the implant through sliding holes which are arranged in a circular way.

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ADVANTAGES

- Minimal outer diameter (Ø 16.5 mm)
- Slight panel thickness / panel height 2.8 mm
- Rotatory compression holes
- Fixable with 8 screws
- Narrow screw diameter (Ø 2.0 mm)
- Self-cutting screw thread
- Variable screw angle (+/- 8°)
- Biocompatible titanium alloy

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Step 1

- Scaphoid resection
- Preparing of joint surfaces and repositioning
- Pre-fixation with Kirschner wires

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FIXATION OF SMALL BONES

The PEEK Kirschner wire applicator enables the implantation of a wire within the scope of an osteosynthesis by single-handed operation. A stainless steel type for sterilizing is also available (KWA).

The balance between function, form and material is a deciding aspect specifically in the medical field. The Kirschner wire applicator combines these elementary qualities into a unique medical instrument.

The two-piece instrument can be opened by pressure relief during insertion of wire, pulled back, thereafter closed anew, whereby the wire is gradually inserted into the bone up to the desired position.

For easier insertion and enhanced stability of rotation the wire tip can be pre-bent by means of an integrated bending device. The inside of the handle is fitted with a protective device so as to protect the surgeon's hand against improper handling.

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ADVANTAGES

- Single-handed operation
- Ergonomically shaped handle
- Self-explanatory operability
- Application of various wire sizes (Ø 0.8-1.6 mm)
- Integrated gauge to control the bending angle
- Sterile packing
- Single-use type
- Material: PEEK with carbon fiber

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For easier insertion of the Kirschner wire into the opened bone the tip of the wire is bendable (Fig.1). Especially for this purpose there is an integrated bending device as well as a bending control function on the wire applicator (Fig.2).

Abb. 1 Manuelles Vorbiegen des Kirschner-Drahtes
Fig. 1 Manual pre-bending of the Kirschner wire

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The Kirschner wire is inserted into the instrument in such a way that the bent wire tip protrudes from the wire applicator.

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Gradual insertion of the wire tip into the bone by opening and closing the wire applicator.

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Removal of the protruding wire end so as to avoid irritation of surrounding tissue.

Artikelnummer	Bezeichnung – Description	
480-155-001	Single-use-Ausführung (PEEK) – Single-use-type (PEEK)	
480-155-002	Stahl-Ausführung (KWA) – Steel-type (KWA)	
480-155-003	Vorstecher – Picker	
380-000-003	Tray mit Abdeckung (KWA), leer – Tray with Cover (KWA), empty	

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Our Kirschner wire consists of 1.4441 implant steel.

Micromed Kirschner wires are generally delivered non-sterile and each packaging unit contains 10 pieces.


Our product line includes 2 variations of wires. Variation 1 Trokar - round and variation 2 Trokar - Trokar.

Artikelnummer	Trokar - Rund 10 cm
481-006-010	Ø 0,6 x 10 cm
481-008-010	Ø 0,8 x 10 cm
481-010-010	Ø 1,0 x 10 cm
481-012-010	Ø 1,2 x 10 cm
481-012-510	Ø 1,25 x 10 cm
481-014-010	Ø 1,4 x 10 cm
481-015-010	Ø 1,5 x 10 cm
481-016-010	Ø 1,6 x 10 cm
481-018-010	Ø 1,8 x 10 cm
481-020-010	Ø 2,0 x 10 cm
481-022-010	Ø 2,2 x 10 cm
481-025-010	Ø 2,5 x 10 cm
481-030-010	Ø 3,0 x 10 cm

Artikelnummer	Trokar - Rund 15 cm
481-006-015	Ø 0,6 x 15 cm
481-008-015	Ø 0,8 x 15 cm
481-010-015	Ø 1,0 x 15 cm
481-012-015	Ø 1,2 x 15 cm
481-012-515	Ø 1,25 x 15 cm
481-014-015	Ø 1,4 x 15 cm
481-015-015	Ø 1,5 x 15 cm
481-016-015	Ø 1,6 x 15 cm
481-018-015	Ø 1,8 x 15 cm
481-020-015	Ø 2,0 x 15 cm
481-022-015	Ø 2,2 x 15 cm
481-025-015	Ø 2,5 x 15 cm
481-030-015	Ø 3,0 x 15 cm

Artikelnummer	Trokar - Trokar 10 cm
482-006-010	Ø 0,6 x 10 cm
482-008-010	Ø 0,8 x 10 cm
482-010-010	Ø 1,0 x 10 cm
482-012-010	Ø 1,2 x 10 cm
482-012-510	Ø 1,25 x 10 cm
482-014-010	Ø 1,4 x 10 cm
482-015-010	Ø 1,5 x 10 cm
482-016-010	Ø 1,6 x 10 cm
482-018-010	Ø 1,8 x 10 cm
482-020-010	Ø 2,0 x 10 cm
482-022-010	Ø 2,2 x 10 cm
482-025-010	Ø 2,5 x 10 cm
482-030-010	Ø 3,0 x 10 cm

Artikelnummer	Trokar - Trokar 15 cm
482-006-015	Ø 0,6 x 15 cm
482-008-015	Ø 0,8 x 15 cm
482-010-015	Ø 1,0 x 15 cm
482-012-015	Ø 1,2 x 15 cm
482-012-515	Ø 1,25 x 15 cm
482-014-015	Ø 1,4 x 15 cm
482-015-015	Ø 1,5 x 15 cm
482-016-015	Ø 1,6 x 15 cm
482-018-015	Ø 1,8 x 15 cm
482-020-015	Ø 2,0 x 15 cm
482-022-015	Ø 2,2 x 15 cm
482-025-015	Ø 2,5 x 15 cm
482-030-015	Ø 3,0 x 15 cm

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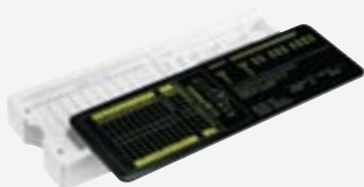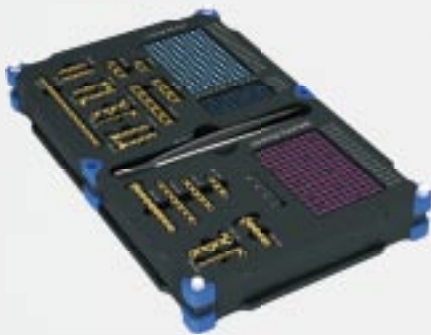
381-000-005

Handplatten-Schrauben-Set
Plate screw set

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Components

*Forceps 14.5 cm
 10 locking screws per type
 One plate each per type
 Tray with cover*

HANDSET

Lockable screw and plate set with instruments for fracture fixation of the hand.

The Micromed handset offers the surgeon the option of a special selection of plates and screws which remain at a stable angle, in order to attend to occurring small bone fractures.

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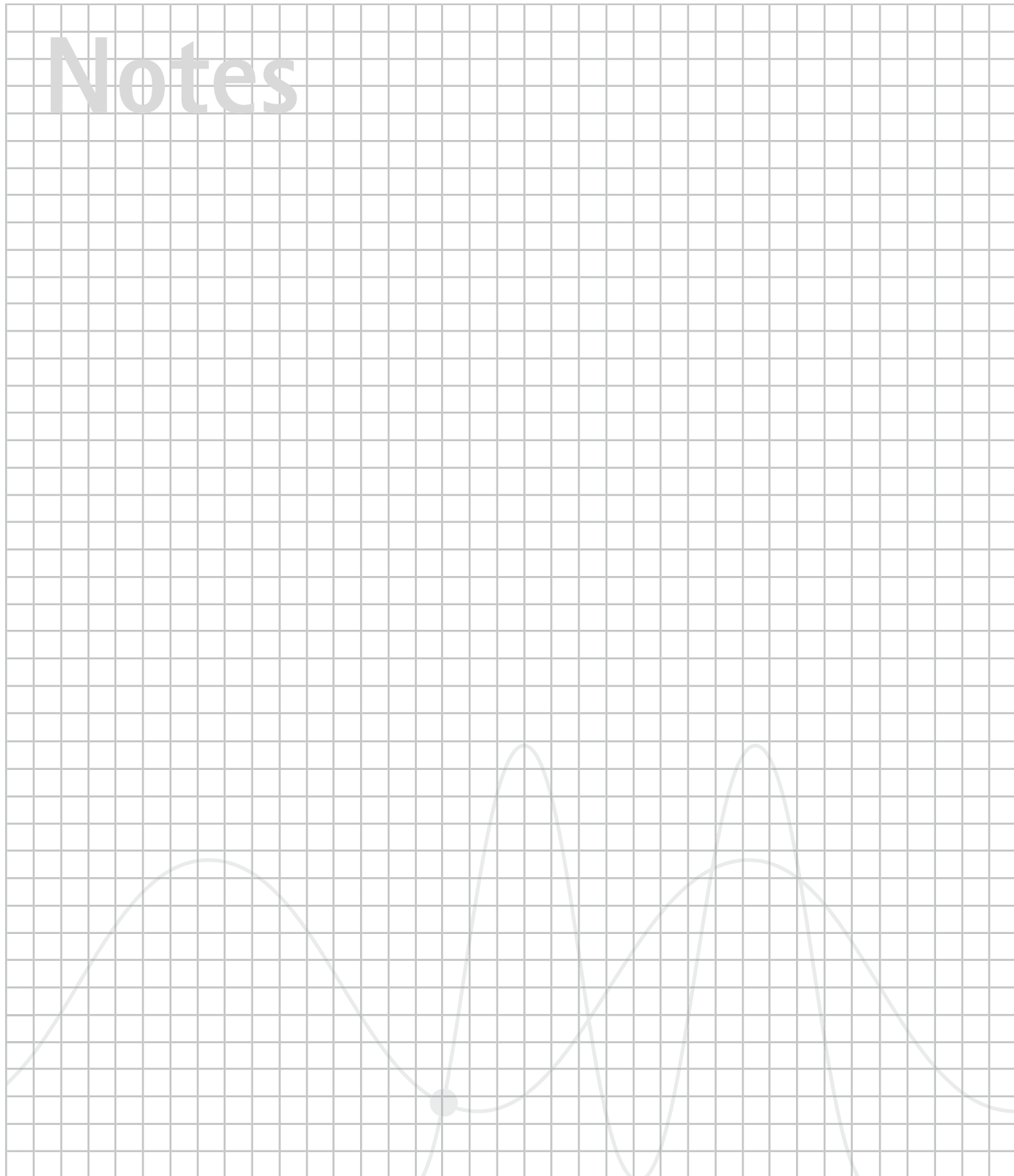
Components

*Screwdriver handle, AO-DK
 Screwdriver Bit T6
 Drill jig
 Drill Ø 1.5 und Ø 1.8
 Depth measuring gauge
 Plate cutter
 Plate bending pliers
 Tray with cover*

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Micromed Medizintechnik GmbH